KwikVial Container and Accessories

Amber plastic vials with durable heat-sealed foil lids, used for unit dose packaging, delivery and administration of oral liquid medications. Plastic tray keeps vials clean, separated and in fill position. KwikVial Filling Extension attaches to any male luer fitting to facilitate vial filling. Perforated KwikVial Labels available for vial identification.

Specifications

- Vial: high-density polyethylene (HDPE)
- Sealing Plug: polypropylene (PP)
- Foil lid: 0.1mm thick foil laminate
- Amber colorant: inert pigment – conforms to USP light transmission requirements
- Filling Extension: poly(vinyl chloride (PVC)
- Non-sterile
- Made in the USA

Contraindications:

None when used as described in product insert.

Instructions for Use

1. Set up KwikVial Trays with open end up. Transparent trays allow visibility for filling and storage.
2. Fill vials to the desired volume, row-by-row.
3. Seal the vials manually by pushing a sealing plug into place in the vial opening. NOTE: Do not push the plug all the way to the bottom of the ball seat. Over-insertion may result in a cracked seat.
4. Or, use the KwikVial Plug Dispenser. Slide open the dispenser top and empty plugs into chamber. Align the dispenser over the vial opening then push the plunger down. The sealing plug will lock into place.
5. Invert the vial tray and remove the plastic lid with a twist-and-lift motion at the offset tabs on opposite corners of the tray.
6. Peel label backing off five labels and place over the foil tops. Labels may also be placed on the vial sides.
7. Dispense KwikVial Containers by separating vials at the perforations.

	Do not reuse. Product reuse can be unsafe because of the risk of: cross-infection, endotoxin reaction, patient injury due to material failure and/or chemical reaction due to cleaning or sterilization agents.		See Instructions for Use – IFU and Manuals
	Discard if packaging is damaged.		No natural rubber latex components

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© Copyright 2018 MedSafety Solutions 53001252 Rev. B