eng Tamper-Resistant Add Port Cap

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MedSafety Solutions
7012 S. Revere Pkwy, Suite 120
Centennial, CO 80112 USA

© Copyright 2020 MedSafety Solutions 53001263 Rev. B

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Tamper-Resistant Add Port Cap

Available in Red and Blue versions to fit a variety of additive ports. These caps are a durable hinged plastic cap that impedes IV bag tampering by restricting port access. When applied, the cap is difficult to remove and prevents needle penetration.

Specifications:

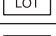
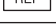
- Non-Sterile
- Single Use
- Made in the USA
- Not made with natural rubber latex
- Material:
 - o Red – HDPE (high-density polyethylene)
 - o Blue – PP (polypropylene)

Indications for Use:
Tamper-Resistant Add Port Cap is indicated for use on IV bag additive port to prevent tampering by restricting port access.

Directions:

1. After additives have been injected, place the Tamper-Resistant Cap over the additive port and snap closed.
2. Verify the cap is fully closed and proper fit.

Warnings and Contraindications:
If the cap cannot be secured (snap closed) or if the additive port diameter is smaller than the cap opening, the Tamper-Resistant Add Port Cap should not be utilized.

SYMBOL GLOSSARY:	
Symbol	Title of Symbol/Definition/Reference
	Do not re-use / Indicates device is intended for one use or for use on a single patient during a single procedure / Ref. 5.4.2 ISO 15223-1:2016
	Consult instructions for use / Indicates the need for the user to consult the instructions for use / Ref. 5.4.3 ISO 15223-1:2016
	CAUTION! Indicates important cautionary information. / Ref. 5.4.4 ISO 15223-1:2016
	Do not use if packaging is damaged / Indicates device should not be used if package is damaged or opened / Ref. 5.2.8 ISO 15223-1:2016
	Manufacturer / Indicates the medical device manufacturer name & address / Ref. 5.1.1 ISO 15223-1:2016
	Batch code / Indicates the manufacturer's batch code / Ref. 5.1.5 ISO 15223-1:2016
	Catalogue Number / Indicates the manufacturer's catalogue number / Ref. 5.1.6 ISO 15223-1:2016

MedSafety Solutions is a trademark of MedSafety Solutions LLC, or its subsidiaries.

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MedSafety Solutions er et varemærke tilhørende MedSafety Solutions LLC eller dets datterselskaber.